



AMD Technologies, Inc.
 218 Bronwood Avenue
 Los Angeles, CA 90049-3104


T 310.470.8900
 T 800.423.3535
 F 310.471.8900

W digitalams.com
 W ams4illuminators.com
 E info@digitalams.com

Mobile Cassette Holders

PRODUCT: Mobile Cassette Holders
 SERIES: Cassette Products
 MODEL(S): 10900-DR, 10900-E, 10900-L, 10900-S

Componets and Assemebly

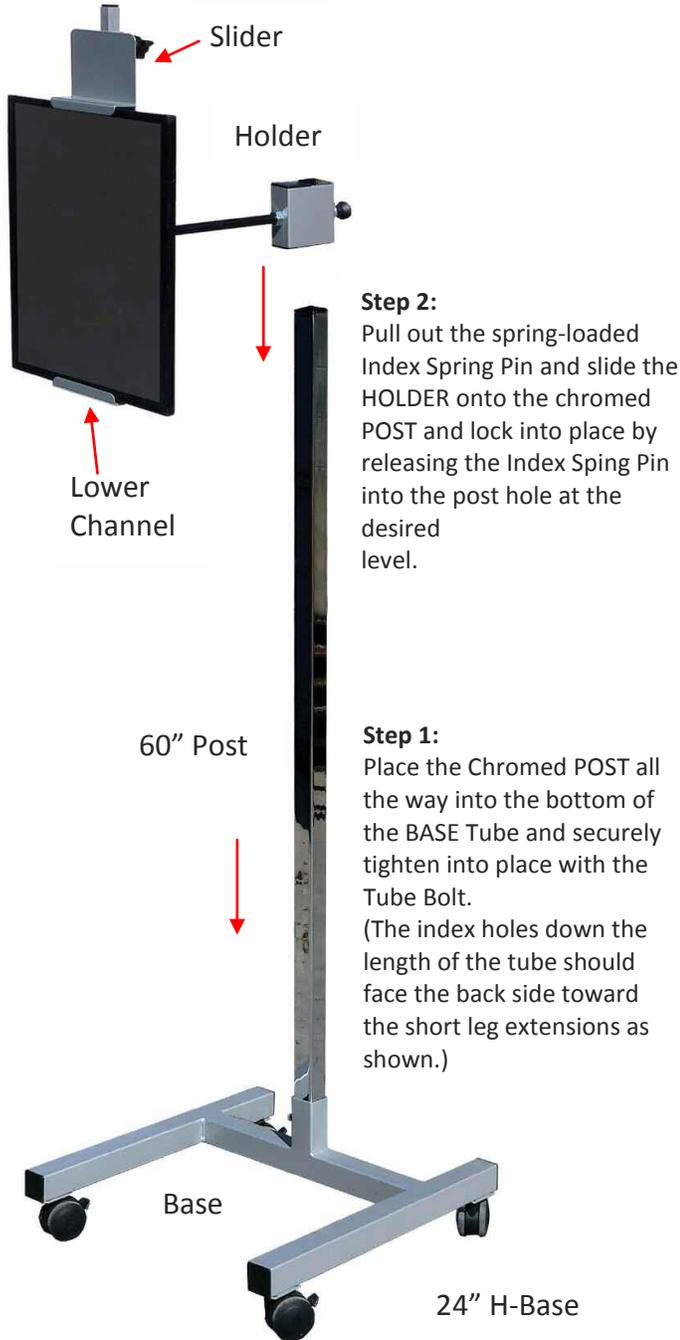


Instructions for Use

Model 10900-S

(standard tilt & rotate)


- (a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.
- (b) Place receptor onto Lower Channel and position the Slider onto the top of the receptor and secure with Knob-b.
- (c) Loosen Knob-c and rotate the HOLDER into the desired position and re-tighten Knob-c.

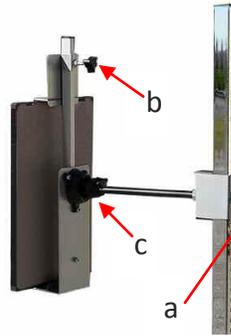

Model 10900-E

(extension arm tilt & rotate)
 (a), (b), (c) See MCH-33 above.


- (d) Pull the Holder away from the Post to extend the Holder closer the patient. Push the Holder back to the Post when not extended and it should gently lock into place.

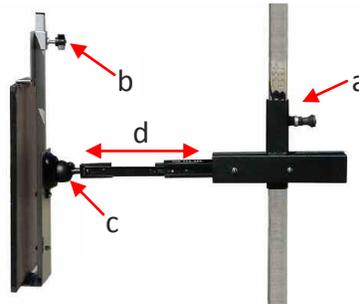

Model 10900-L

(tall receptor holder)

- (a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.
- (b) Place receptor onto Lower Channel and position the Slider a little over halfway up of the receptor and secure with Knob-b.
- (c) Move the lateral sliders onto the sides of the receptor and tighten Knob-c (x2).

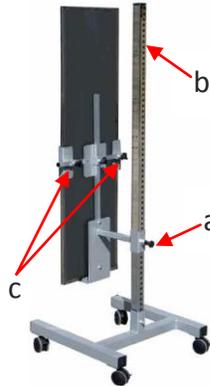

Model 10900-DR

(DR-Receptor holder)

- (a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.
- (b) Place DR-Receptor onto Lower Channel and position the Slider onto the top of the plate and secure with Knob-b.

